

Super-Bond

Universal Ceramic Primer

Designed for All Types of Ceramic and Zirconia
Assured Bonding Performance
Excellent Quality Stability




 **SUN MEDICAL**
www.sunmedical.co.jp

Super-Bond Universal Ceramic Primer

A priming agent for ceramic surface. Application of Super-Bond Universal Ceramic Primer on a ceramic surface before bonding with a dental adhesive resin cement will enhance the bond strength and durability.

Bond Strength with Zirconia

Tensile Bond Strength with Different Zirconia * (MPa)

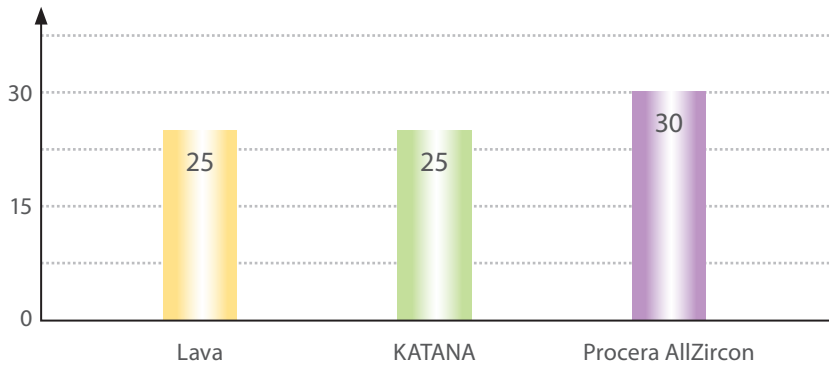


* Tested after thermocycled 10,000 times (5-55°C)
Resin Cement: Super-Bond

Assures to bond zirconia firmly to tooth using Super-Bond

Bond Strength with Porcelain

Tensile Bond Strength with Different Porcelain * (MPa)

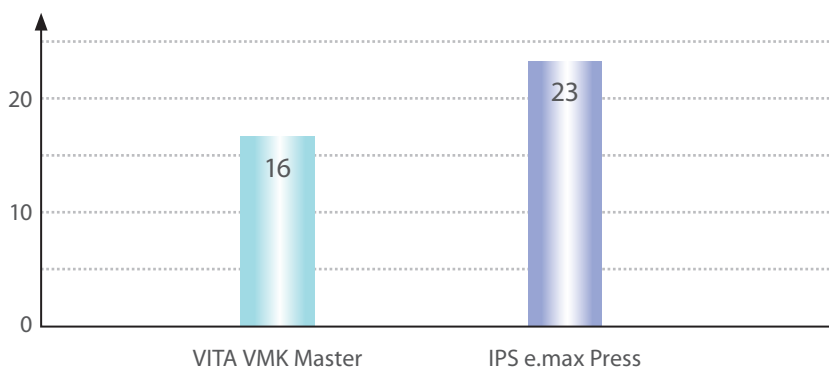

* Tested after thermocycled 10,000 times (5-55°C)
Resin Cement: Super-Bond

Guarantees a reliable adhesion to porcelain

Clinical Case

A Zirconia Bridge




Application of Super-Bond Universal Ceramic Primer



Dispense Liquid A and Liquid B, mix and apply to the inner surface of the zirconia bridge

Application of Resin Cement

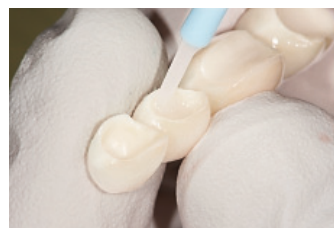

Use Super-Bond to bond the bridge

After Seating

