Metafil CX

Type	Conventional halogen	Fast halogen	Plasma arc	LED			
Curing time (sec)	30	5	10	3	20	30	
Depth of cure (mm)	A1, A2, A3, A3.5, A4						
	B2	2.5	2.5	3.0	2.5	2.0	2.5
	C2, C3						
	Incisal						
Universal	2.0	2.0	2.5	2.0	—	2.0	

- For best handling, use Metafil CX at room temperature.
- If the cavity preparation is large and/or deep, incremental filling techniques should be employed.
- If dispensed material is not used immediately, cover it to protect the material from ambient light preventing premature polymerization.

Polishing :

Metafil CX will polish quickly and easily to a high luster. Use of most discs or polishing paste systems are acceptable.

As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.

EN

INSTRUCTIONS

IMPORTANT :
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Resin-based Dental Restorative Material

Metafil CX

FOR DENTIST USE ONLY

1. What is Metafil CX?

Metafil CX is a visible light cure, tooth shade composite resin using an organic filler which is reactive with the polymer matrix. Metafil CX is an anterior composite. Do not use it for an occlusal. The inorganic filler loading is 27.2 vol% with a particle size range of 0.02 to 1 µm.

Indications:

1. Direct restorative for Class III, IV, and V cavities.
2. Direct restorative for wedge-shaped defects and root surface cavities.

2. Types

Individual Syringes : Containing 2.5mL (3.5g) each
Shades : A1, A2, A3, A3.5, A4,
B2,
C2, C3,
Incisal,
Universal

3. Composition

Methacrylate monomers, Organic filler, Inorganic filler, Aromatic amine, etc.

4. Precautions

- This product contains substances that may cause allergic reactions.
- Metafil CX should not be used either by clinicians or on patients who are methacrylate monomer-sensitive.
- Avoid contact of Metafil CX with soft tissues, skin or eyes. Contaminated skin or mucosa with Metafil CX should be wiped off immediately with alcohol and then rinsed thoroughly with running water. If Metafil CX enters eye, rinse the eye with running water immediately. The patients should then be examined by an ophthalmologist.
- Dentist should wear rubber or PVC dental gloves.
- If necessary, the pulp must be indirectly capped locally for deep cavities with only a minimal layer of dentin.
- Avoid accidental ingestion while applying and rinsing this material. If ingested, the patient must seek medical attention.
- Use a dental rubber dam to isolate the treatment area from contamination.
- Use Metafil CX only for the applications recommended in this leaflet.

5. Storage

Store Metafil CX at a room temperature between 1-30°C. Avoid high temperatures above 30°C, high humidity and direct sunlight. Refrigeration is not necessary. However, if stored in a refrigerator, then allow the material to warm to room temperature prior to use.

Shelf life: 3 years in a room temperature. See outer package for expiration date. Do not use after expiration date.

6. Disposal

Dispose of empty containers, etc. in accordance with local regulations.

7. How to use Metafil CX

Adhesive Bonding System :

Apply light-cured bonding system according to the manufacturer's instructions. We recommend the use of Hybrid Bond from Sun Medical Co., Ltd.

Placement and Curing :

- A. Dispense Metafil CX on to a mixing pad and promptly recap the syringe.
- B. Place dispensed material into prepared cavity previously treated with a dentin adhesive.
- C. Light-cure using dental visible light-curing device according to the manufacturer's instructions.

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