Light Fix

Dental Splinting Material

EN

INSTRUCTIONS

IMPORTANT :
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Dental Splinting Material

Light Fix

FOR DENTIST USE ONLY

1. What is Light Fix?

Light Fix is a light-cure adhesive resin for splinting of mobile teeth and bonding of orthodontic attachments. Paste of Light Fix can achieve bond to etched* enamel without priming.

Indications:

- Splinting of mobile teeth such as traumatized tooth or mild-moderate periodontitis.
- Bonding of orthodontic attachments to teeth surface or dental restorations.
- *Etchant Gel is used for etching on teeth or dental restorations.

2. Contents of the kit

2.1 Light Fix Kit:

Paste (Clear)	2 (1.5mL)
19G Needle Tip	10 (with 2 grey Needle Caps)
Etchant Gel	1 (3mL)
23G Needle Tip	5 (with 1 blue Needle Cap)

Attached documents: Instructions / Pictorial Instruction Card

2.2 Refills:

Paste (Clear)	1 (1.5mL)
Paste (Tooth color)	1 (1.5mL)
19G Needle Tip	20 (with 1 grey Needle Cap)
Etchant Gel	1 (3mL)
23G Needle Tip	20 (with 1 blue Needle Cap)

3. Compositions

Paste (Clear, Tooth color): Methacrylate monomers, Acrylate monomers, Initiator and others.
 Etchant Gel: Phosphoric acid, thickener, water and others.

4. Precautions

4.1 Safety

This product contains substances that may cause allergic reactions. Paste of Light Fix should be contraindicated for use by clinicians or on patients who have a history of hypersensitivity to acrylate or methacrylate monomers. If signs of irritation such as redness appear, stop using it immediately and consult a physician.

1) Precautions for use

1. Avoid mixing with other materials.
2. Proper use of eye protection is recommended when using light irradiation devices.
3. Do not use or store near heat or open flame.
4. If accidentally swallowed during use, immediately have the patient gargle with water, consult a physician if necessary.
5. Needle Tip (19G and 23G) is one-time use only.

2) Important Precautions

A. Sensitivity

- In application of the material, hypersensitivity reactions such as rash or dermatitis may occur.
- Discontinue the use immediately if signs of hypersensitivity reaction such as rash, eczema, redness, tumor, swelling, irritation, eruption or numbness appear. See a physician.
- Make the best clinical judgement on selecting the treatment and materials, as each patient and case has its unique requirement.

B. Avoid contacts to eye, mucosa and skin

- Clinicians should use gloves to avoid direct contact with the material, especially uncured paste. If signs of hypersensitivity reaction appear, see a physician.
- Material contacted on the mucosa may cause inflammation such as swelling of the lips or white sore at the corner of the mouth. Use a rubber dam etc. for avoiding contact with mucosa such as gingiva and lips etc. If contacted with mucosa or skin, wipe off the paste with alcohol-impregnated cotton pledgets immediately and rinse with copious amounts of running water thoroughly. In case of eye contact, flush immediately with copious amounts of running water and consult an ophthalmologist if necessary. Corrosive phosphoric acid contained in Etchant gel may cause swelling or blister if the residue is left on soft tissue or skin.

4.2 Storage

Please take the following precautions for the best performance.

① Storage conditions

Store Paste (Clear/Tooth color) and Etchant Gel in a cool (1°C - 30°C) and dark location. High temperature, high humidity and direct sunlight will shorten their shelf-life. Store secured with access only to the dental professionals.

② Contamination

To avoid contamination, do not mix the components with other materials. Do not re-use Needle Tip or recap with a Needle Cap (grey, blue). Seal the containers with the original cap immediately after use.

③ Shelf life

Paste (Clear/Tooth color) : 26 months at room temperature
 Etchant Gel : 36 months at room temperature

*See outer package for expiration date. Do not use after expiration date.

4.3 Disposal

Dispose empty containers or packages according to the local waste regulations.

As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.

5. How to Use

Splinting of mobile teeth

1. Surface preparation

a. Teeth

- Cleaning of the teeth surface

Remove contaminants and stains using a polishing bur or oil-free, fluoride-free pumice to expose fresh tooth surface. Rinse thoroughly and dry. A rubber dam is highly recommended to reduce the chance of contamination by saliva, breath humidity or blood.

- Etching

Remove the syringe cap of Etchant Gel and attach a 23G Needle Tip (blue) to the syringe. Apply the Etchant Gel to the tooth surface and leave it for 30 sec. Then, rinse with water and dry thoroughly. Keep it isolated from contaminants.

b. Metal surface

- Rinse the metal surface thoroughly and dry. Applying the Etchant Gel helps remove surface contaminants. Air-abrade the metal surface with 50-micron aluminum oxide.

- If the surface is precious metal, apply a metal primer such as V-PRIMER on the metal surface to be bonded in order to increase bond strength. V-PRIMER is sold separately by Sun Medical Co., Ltd. For application details refer to the instructions for the metal primer.

c. Ceramic (porcelain/zirconia) surface

- Clean the ceramic (porcelain/zirconia) surface with abrasive wheels and points or a prophylactic cup with fluoride-free, oil-free pumice. Rinse thoroughly with water and dry. Applying the Etchant Gel helps remove surface contaminants.

- Apply a ceramic primer such as Porcelain Liner M / Super-Bond Universal Ceramic Primer on the ceramic (porcelain/zirconia) surface to be bonded to increase bond strength. Porcelain Liner M / Super-Bond Universal Ceramic Primer is sold separately by Sun Medical Co., Ltd. For application details refer to the instructions for the ceramic primer.

2. Application of Paste

Remove the syringe cap of Paste and attach a 19G Needle Tip (black) to the syringe. Dispense the Paste to the prepared surface from both lingual and labial/buccal sides. Remove the excess with scalers or explorers.

3. Curing

Irradiate light on the Paste from both lingual and labial/buccal sides using dental visible light-curing device. Refer to following table for the curing time. Curing time can vary for each device, irradiate adequately as indicated.

Curing time for type of light source

Type	Curing time
LED	10 sec
Halogen	20 sec
Plasma Arc	6 sec

LED: $\geq 1000\text{mW/cm}^2$

Halogen: $\geq 400\text{mW/cm}^2$

Plasma arc: $\geq 1840\text{mW/cm}^2$

(For the light intensity, refer to the IFU of the dental curing unit.)

4. Adjust Occlusion

Check and adjust occlusion carefully with conventional stones and wheels for resins.

5. Finish and Polish

Finish and polish with conventional methods for resin. If no adjustment and polishing are indicated, wipe off an unpolymerized surface layer with alcohol-impregnated cotton pledgets.

Bonding of an orthodontic attachment to teeth or dental restorations.

1. Surface preparation

Refer to the surface preparation of Splinting of mobile teeth.

2. Application of Paste

Remove the syringe cap of Paste and attach a 19G Needle Tip (black) to the syringe. Dispense a small amount of Paste onto the bracket or tube base.

3. Placement and Curing

Press the bracket or base onto the prepared surface. Irradiate light on the Paste according to the following table.

Type of attachment	Light curing condition
Metal bracket or Tube	Both mesial and distal sides
Ceramic or plastic bracket	Perpendicular to bracket surface

* Before curing, Paste is low in viscosity and flowable. Light-cure the attachment one by one while securing the attachment on a tooth, for direct bonding system.

4. Remove the excess Paste around the bracket/tube base. If there is no excess, wipe off an unpolymerized surface layer on the cured Paste with alcohol-impregnated cotton pledgets.

5. Place the orthodontic wires.

Precautions on how to use

Read all instructions thoroughly before use.

- 1) Plaque and contamination on tooth surface compromise bonding strength. Remove plaque and contaminants mechanically before application.
- 2) Use a fluoride-free polishing paste to clean tooth surface. Isolate teeth to avoid re-contamination.
- 3) Bond strength to the prosthesis may not be sufficient even after use of recommended primers. Adjustment of Paste application, such as bonding to increased surface area, may be necessary for increasing durability.
- 4) Sufficient clinical outcome might not be achieved with some cases with severe mobility, broad intra-dental width, or strong occlusal stress. For those, incorporating fiber or wire reinforcements should be considered.
- 5) Adjust occlusion to avoid excess stress on the splinted area.
- 6) Make the light irradiator close to the Paste as much as possible when irradiating light.
- 7) Check the performance of the light curing device. Insufficient polymerization from the compromised irradiator may cause the lower adhesive strength. Clean and exchange the light bulb periodically.
- 8) Consistency of Paste can change by the temperature. Use at the recommend room temperature (15~25°C) to yield the same consistency.
- 9) Use provided 19G or 23G Needle Tip included in the kit.
- 10) Needle Cap (grey, blue) is to avoid leakage of the paste due to a residual pressure and should be detached just before use.
- 11) Attach Needle Tip to Paste and Etchant Gel and twist firmly until it is tight.
- 12) Remove Needle Tip after use, and cap the syringe with syringe cap (black) immediately.
- 13) Do not press the plunger with a sudden strong force. Etchant Gel might be splash out.
- 14) To avoid cross-infection, use a new Needle Tip for each patient.

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2.1 Light Fix Kit:

Paste (Clear) -----	2 (1.5mL)
19G Needle Tip -----	10 (with 2 grey Needle Caps)
Etchant Gel -----	1 (3mL)
23G Needle Tip -----	5 (with 1 blue Needle Cap)

Attached documents: Instructions / Pictorial Instruction Card

2.2 Refills:

Paste (Clear) -----	1 (1.5mL)
Paste (Tooth color) -----	1 (1.5mL)
19G Needle Tip -----	20 (with 1 grey Needle Cap)
Etchant Gel -----	1 (3mL)
23G Needle Tip -----	20 (with 1 blue Needle Cap)

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Paste (Clear, Tooth color): Methacrylate monomers, Acrylate monomers, Initiator and others.
Etchant Gel: Phosphoric acid, thickener, water and others.

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