SEcure

Dental Adhesive Resin Cement

EN

INSTRUCTIONS

IMPORTANT :
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Dental Adhesive Resin Cement

SEcure

FOR DENTIST USE ONLY

1. Product overview

SEcure Starter Kit is a self-etching, dual-cure resin cement for onlays, inlays, crowns and bridges. The cement is radiopaque.

The pretreatment with V-PRIMER for precious metals and Porcelain Liner M/Super-Bond Universal Ceramic Primer for porcelain will enhance the bond strength and its durability.

Indication:

SEcure Starter Kit is used for the permanent cementation of indirect restorations:
 - Inlays, onlays, crowns, bridges, laminate veneer and root posts

2. Components

SEcure Starter Kit

- ① SEcure Dual-cure Resin Cement (in double-barrel syringe) -----1x9g
 (Shade : Translucent)
- ② SEcure Self-etching Primer -----1x3mL
- ③ Mixing Tip (X) -----10pcs
- Attached document: Instructions

Sold separately :

- ① SEcure Dual-cure Resin Cement (in double-barrel syringe) ----1x9g
 (Shade : Translucent, White Opaque, Dentin)
- ② Mixing Tip (X) -----10pcs
- SEcure Self-etching Primer-----1x3mL
- Mixing Tip (X) -----30pcs

3. Compositions

SEcure Dual-cure Resin Cement: Methacrylate monomers, Aromatic amine, Inorganic fillers, and the others.

SEcure Self-etching Primer: Methacrylate monomers, Water, Aceton, Aromatic amine, and the others.

4. Precautions

4.1 Safety

This product contains substances that may cause allergic reactions. Read the following points below.

① Sensitivity

SEcure Starter Kit should not be used by clinicians or on patients who are sensitive to methacrylates or resin cements. If signs of irritation such as redness appear, stop using SEcure Starter Kit immediately and consult a physician.

② Avoid contact

Avoid contacting soft tissues, skin or eyes by SEcure Starter Kit components. The use of rubber dam is recommended. Contaminated skin or mucosa should be wiped immediately with alcohol and then rinsed thoroughly with copious amounts of running water. If SEcure Starter Kit components enter the eye, rinse immediately and thoroughly with running water and have the patient examined by an ophthalmologist. Inhalation of a large amount of concentrated vapor from SEcure Self-etching Primer may cause headache: therefore the area should be well ventilated. In case of such severe inhalation, move to fresh air. Clinicians should use dental gloves.

③ Avoid ingestion

Avoid accidental ingestion while applying and rinsing this material. If ingested, the patient must seek medical attention.

④ Pulp protection

If the preparation approaches the pulp, apply a protective liner or base.

⑤ Flammability

The SEcure Self-etching Primer is flammable. Keep away from open flame in use and storage.

⑥ Applications

Only use SEcure Starter Kit for the applications recommended in these instructions.

4.2 To get the best results

① Storage conditions

The kit should be stored **IN A REFRIGERATOR (2 - 10°C)**. High temperature and direct sunlight will shorten their shelf-life.

The product must be kept away from extreme heat, direct sunlight or a flame. The product must be stored in a proper place where only dental practitioners can access.

② Product lifetime

Product lifetime can be shortened depending on the circumstances of use. Read all provided instructions carefully before use.

③ Avoid Contamination

To avoid contamination, do not mix the components with other materials. Firmly recap the containers immediately after use. After the cementing procedure, dispose of any excess of mixed materials and Mixing Tip. Once dispensed do not return materials to the containers.

④ Do not reuse

Mixing Tip is for one-time use only. It is not autoclavable or chemically sterilizable. Dispose it after each patient to avoid cross-contamination.

4.3 Disposal

Dispose of empty containers and packages in accordance with local regulations.

As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.

5. How to use

Note : The use of eugenol-based temporary materials may compromise the polymerization of SEcure Starter Kit.

① Surface preparation

1) Tooth Surface

Clean cavity/preparation, wash with water and dry with oil-free air. (A rubber dam is highly recommended to reduce the chance of contamination by saliva, breath humidity or blood.)

Then apply the SEcure Self-etching Primer to the tooth surface. After 20 seconds, spread the SEcure Self-etching Primer by gentle air blow to make the thin film and looks shiny surface.

2) Restoration Surface

To the extent practically possible, the surface of a restoration to be bonded should be abraded using a sand-blastor or roughened by a round bur. Rinse thoroughly and dry. Then, depending on the restorative material, the roughened and cleaned surface may be treated with a surface modifier, such as Porcelain Liner M/Super-Bond Universal Ceramic Primer for porcelain or V-PRIMER for precious metals, to enhance the bond strength and durability.

② Mixing and application of the cement

- **Important :** Remove sealing cap from the dual-barrel cartridge and express a small amount of base and catalyst until they flow equally from both orifices.
 - Attach and lock a Mixing Tip onto the cartridge and express a small amount of mixed material to confirm good flow and mixture.
 - Apply the mixture to the surfaces of the prepared tooth and/or the restoration.
 - Carefully wipe the orifices and re-seal cartridge taking care to align the V on the cap with the notch on the cartridge collar. This will assure that the proper plugs fit the holes. (If the plugs are reversed, the cement may set in the syringe.)
- Note :** The mixture should be used within 30 seconds of mixing.

③ Cementing, setting and finishing

- Insert the restoration immediately after the application of the cement. After confirming that it is completely seated, hold it under pressure in position until the cement sets.
- To remove the excess cement, irradiate the excess with visible light for 1 - 2 seconds and immediately remove it. Or, alternatively without light, remove the excess on the halfway to setting, specifically 1.5 - 2 minutes after cementing.
- The final setting time is approximately 5 minutes in the mouth, thereafter polish the cement line or adjust occlusion according to normal practices.

Option : Light-setting

If the restoration is translucent like porcelain or cured composite resin, light-setting may be opted. In this case, irradiate visible light first for 1 - 2 seconds, remove the excess cement, and then irradiate the light another 30 seconds for the final setting.

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SEcure Starter Kit

①SEcure Dual-cure Resin Cement (in double-barrel syringe) --- 1×9g

(Shade: Translucent)

②SEcure Self-etching Primer ----- 1×3mL

③Mixing Tip (X) ----- 10pcs

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• ①SEcure Dual-cure Resin Cement (in double-barrel syringe) -- 1×9g
(Shade: Translucent, White Opaque, Dentin)

②Mixing Tip (X) ----- 10pcs

• SEcure Self-etching Primer ----- 1×3mL

• Mixing Tip (X) ----- 30pcs

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SEcure Dual-cure Resin Cement: Methacrylate monomers, Aromatic amine, Inorganic fillers, 기타.

SEcure Self-etching Primer: Methacrylate monomers, Water, Aceton, Aromatic amine, 기타.

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SUN MEDICAL CO.,LTD.