# Zirconia Liner Premium

## INSTRUCTIONS

**IMPORTANT:**  
READ ALL INSTRUCTIONS THOROUGHLY BEFORE USE.  
KEEP THIS LEAFLET AND REFER TO IT PERIODICALLY.

Adhesive Primer for Zirconia Bonding

## Zirconia Liner Premium

FOR DENTIST OR DENTAL TECHNICIAN USE ONLY

### 1. What is Zirconia Liner Premium?

Zirconia Liner Premium is a priming agent for zirconia surface. Application of the product on a zirconia surface before bonding with a dental adhesive resin cement such as Super-Bond will improve the bond strength and durability.

### 2. Contents of the kit

- ① Zirconia Liner Premium ----- 5mL
- ② Plastic Dispensing Dish ----- 5
- ③ Sponge (L-S) ----- 1
- Attached documents
  - Instructions

### 3. Precautions: Read all instructions thoroughly before use.

#### 3.1 Safety Please keep the following precautions for safe use.

##### ① Applications

Use Zirconia Liner Premium only for the applications recommended in this leaflet.

##### ② Past history of sensitivity

Zirconia Liner Premium should not be used by clinicians / technicians or on patients who are methacrylic monomer-sensitive.

##### ③ Symptomatic irritation

Cease using Zirconia Liner Premium immediately, if signs of irritation such as rashes appear, and see a doctor.

##### ④ Avoid contact

Avoid contact with soft tissue, skin or eyes. A rubber dam is recommended for intraoral use. Contaminated skin or mucosa should be wiped off immediately with alcohol and then thoroughly rinsed with running water. If Zirconia Liner Premium enters the eye, immediately rinse thoroughly with running water. The patient should be examined by an ophthalmologist. Be sure to wear dental gloves.

##### ⑤ Give care to flammability

Zirconia Liner Premium is flammable. Do not store where it may be exposed to open flame.

#### 3.2 Storage

##### ① Storage conditions

Store Zirconia Liner Premium in a cool, dark location. High temperature, high humidity and direct sunlight will shorten its shelf-life.

##### ② Volatility

Zirconia Liner Premium is highly volatile. Recap the bottle immediately.

##### ③ Contamination

Plastic Dispensing Dish is disposable. Do not reuse, and avoid using with other materials.

**As in any dental treatment, the patient's individual constitution and the unique requirements of clinical case at hand must be considered before selecting materials and conditions for use.**

### 4. How to use Zirconia Liner Premium

#### ① Pretreatment of the zirconia surface

Remove contaminants and stains from the zirconia surface. Depending on the case at hand, clean by use of abrasive wheels and points or a prophyl cup with fluoride-free, oil-free pumice, then rinse thoroughly with water and dry.

\*Applying the Red Activator/ Enamel Etchant Gel will help remove surface contaminants.

\*Ultrasonic cleaner is also effective.

#### ② Application of Zirconia Liner Premium

Dispense 1 or 2 drops of Zirconia Liner Premium to a well of a Plastic Dispensing Dish. Using an application sponge in the kit, apply the Zirconia Liner Premium to the zirconia surface to be bonded.

\*Blow lightly if the liquid remains in a dent.

\*Application of the Zirconia Liner Premium to tooth or metal surface will not affect the adhesion.

#### ③ Bonding Operation

Apply the adhesive resin cement following the instructions.

\*Carefully avoid contamination of the applied surface, and apply the resin cement quickly.

### 5. Clinical Applications

- ① Restoration using zirconia
  - Adhesion of Inlays and onlays, crowns, bridges
- ② Adhesion of orthodontic bracket to zirconia
- ③ Zirconia implant and zirconia crown for dental implant

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